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**COVID-19 Omicron Variant Test Report**

(2022) FT category No.3

**Product Name:** RABLISS KO264 COVID-19 AG TEST KIT

**Test Category:** Clinical Experiment

**Institute:** Hwason Medical Laboratory

**Test report of COVID-19 Omicron variant  
for  
RABLISS KO264 COVID-19 AG TEST KIT**

**Institute:** Hwason Medical Laboratory

**Assay for evaluation:** RABLISS KO264 COVID-19 AG TEST KIT

**Manufacturer:** KOBAYASHIYAKUHN CO.,LTD

**Date:** 2022/10/28

## **Clinical Test Report**

### **1. Abstract**

**1 Purpose:** To evaluate the clinical performance of RABLISS KO264 COVID-19 AG TEST KIT from KOBAYASHIYAKUHIN CO.,LTD to COVID-19 Omicron variant BA.5.2/BA.5.2.1/ BQ.1.1/BF.7/BA.2.76/XBB by studies using the simulated Omicron recombinant protein samples.

**2 Statistics:** The clinical performance of RABLISS KO264 COVID-19 AG TEST KIT was determined by testing with simulated Omicron recombinant protein samples. Ten Omicron recombinant protein samples, including Omicron BA.5.2/BA.5.2.1/ BQ.1.1/BF.7/BA.2.76/XBB ,were purchased from two companies respectively.

**3 Conclusion:** It was proved that the RABLISS KO264 COVID-19 AG TEST KIT from KOBAYASHIYAKUHIN CO.,LTD. can effectively detect the existence of Omicron BA.5.2/BA.5.2.1/ BQ.1.1/BF.7/BA.2.76/XBB.

## Clinical Test Report

### 2. Test Design

A total of 6 Omicron recombinant protein(10 samples of each protein), purchased from two companies (Omicron BA.5.2, Omicron BA.5.2.1, Omicron BQ.1.1 samples from Fapon Biotech Inc. and Omicron BF.7, Omicron BA.2.76 , Omicron XBB samples from Sino Biological Inc.), were used to evaluate the clinical performance of RABLISS KO264 COVID-19 AG TEST KIT. The 60 samples were kept in transport medium and shipped to laboratory for testing directly with the RABLISS KO264 COVID-19 AG TEST KIT.

### 3. Result analysis

Omicron variant	Manufacturer	Omicron recombinant protein sample No.	Can be checked out
Omicron BA.5.2	Fapon Biotech Inc.	B1	<input type="radio"/> Yes <input type="checkbox"/> No
		B2	<input type="radio"/> Yes <input type="checkbox"/> No
		B3	<input type="radio"/> Yes <input type="checkbox"/> No
		B4	<input type="radio"/> Yes <input type="checkbox"/> No
		B5	<input type="radio"/> Yes <input type="checkbox"/> No
		B6	<input type="radio"/> Yes <input type="checkbox"/> No
		B7	<input type="radio"/> Yes <input type="checkbox"/> No
		B8	<input type="radio"/> Yes <input type="checkbox"/> No
		B9	<input type="radio"/> Yes <input type="checkbox"/> No
		B10	<input type="radio"/> Yes <input type="checkbox"/> No
Omicron BA.5.2.1	Fapon Biotech Inc.	X1	<input type="radio"/> Yes <input type="checkbox"/> No
		X2	<input type="radio"/> Yes <input type="checkbox"/> No
		X3	<input type="radio"/> Yes <input type="checkbox"/> No
		X4	<input type="radio"/> Yes <input type="checkbox"/> No
		X5	<input type="radio"/> Yes <input type="checkbox"/> No
		X6	<input type="radio"/> Yes <input type="checkbox"/> No
		X7	<input type="radio"/> Yes <input type="checkbox"/> No
		X8	<input type="radio"/> Yes <input type="checkbox"/> No
		X9	<input type="radio"/> Yes <input type="checkbox"/> No
		X10	<input type="radio"/> Yes <input type="checkbox"/> No

**Clinical Test Report**

Omicron BQ.1.1	Fapon Biotech Inc.	A1	<input type="radio"/> Yes <input type="checkbox"/> No
		A2	<input type="radio"/> Yes <input type="checkbox"/> No
		A3	<input type="radio"/> Yes <input type="checkbox"/> No
		A4	<input type="radio"/> Yes <input type="checkbox"/> No
		A5	<input type="radio"/> Yes <input type="checkbox"/> No
		A6	<input type="radio"/> Yes <input type="checkbox"/> No
		A7	<input type="radio"/> Yes <input type="checkbox"/> No
		A8	<input type="radio"/> Yes <input type="checkbox"/> No
		A9	<input type="radio"/> Yes <input type="checkbox"/> No
		A10	<input type="radio"/> Yes <input type="checkbox"/> No
Omicron BF.7	Sino Biological Inc.	C1	<input type="radio"/> Yes <input type="checkbox"/> No
		C2	<input type="radio"/> Yes <input type="checkbox"/> No
		C3	<input type="radio"/> Yes <input type="checkbox"/> No
		C4	<input type="radio"/> Yes <input type="checkbox"/> No
		C5	<input type="radio"/> Yes <input type="checkbox"/> No
		C6	<input type="radio"/> Yes <input type="checkbox"/> No
		C7	<input type="radio"/> Yes <input type="checkbox"/> No
		C8	<input type="radio"/> Yes <input type="checkbox"/> No
		C9	<input type="radio"/> Yes <input type="checkbox"/> No
		C10	<input type="radio"/> Yes <input type="checkbox"/> No
Omicron BA.2.76	Sino Biological Inc.	D1	<input type="radio"/> Yes <input type="checkbox"/> No
		D2	<input type="radio"/> Yes <input type="checkbox"/> No
		D3	<input type="radio"/> Yes <input type="checkbox"/> No
		D4	<input type="radio"/> Yes <input type="checkbox"/> No
		D5	<input type="radio"/> Yes <input type="checkbox"/> No
		D6	<input type="radio"/> Yes <input type="checkbox"/> No
		D7	<input type="radio"/> Yes <input type="checkbox"/> No
		D8	<input type="radio"/> Yes <input type="checkbox"/> No
		D9	<input type="radio"/> Yes <input type="checkbox"/> No
		D10	<input type="radio"/> Yes <input type="checkbox"/> No
Omicron XBB	Sino Biological Inc.	E1	<input type="radio"/> Yes <input type="checkbox"/> No
		E2	<input type="radio"/> Yes <input type="checkbox"/> No
		E3	<input type="radio"/> Yes <input type="checkbox"/> No
		E4	<input type="radio"/> Yes <input type="checkbox"/> No
		E5	<input type="radio"/> Yes <input type="checkbox"/> No
		E6	<input type="radio"/> Yes <input type="checkbox"/> No
		E7	<input type="radio"/> Yes <input type="checkbox"/> No
		E8	<input type="radio"/> Yes <input type="checkbox"/> No
		E9	<input type="radio"/> Yes <input type="checkbox"/> No
		E10	<input type="radio"/> Yes <input type="checkbox"/> No

## **Clinical Test Report**

### **4. Conclusion**

In conclusion, according to laboratory clinical results, RABLISS KO264 COVID-19 AG TEST KIT from KOBAYASHIYAKUHIN CO.,LTD has good detection capabilities for Omicron BA.5.2/BA.5.2.1/ BQ.1.1/ BF.7/ BA.2.76/XBB.